
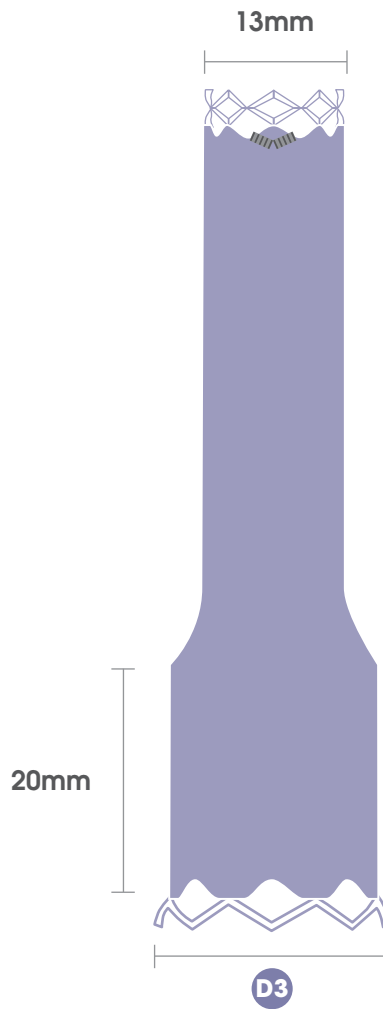



# ALTURA PRODUCT SIZE GUIDE

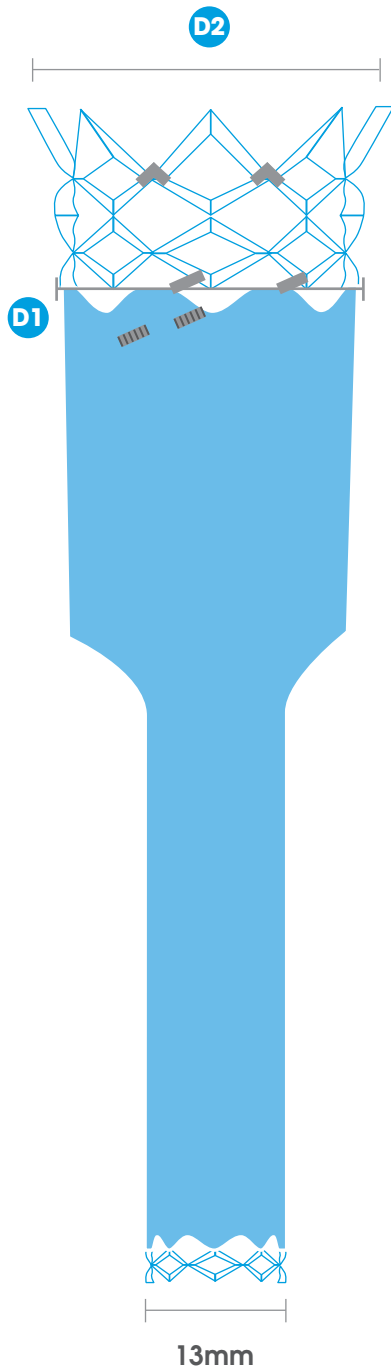

Product Code	D1 Endograft Diameter (mm)	*Treatable Vessel Lumen Diameter (mm)	D2 Supraceliac Stent Diameter (mm)
01-AA-24-090	24	18-22	29
01-AA-27-090	27	21-25	33
01-AA-30-090	30	24-28	36

Product Code	D3 Iliac Endograft Diameter (mm)	*Treatable Vessel lumen Diameter (mm)	Length of Distal Flare
01-AI-13-065	13	8-11	N/A
01-AI-17-065	17	11-15	20
01-AI-21-065	21	15-18	20

\*Diameter measurements should be determined by the vessel lumen  
 The Altura Aortic Endograft length is standard across the product range at 90mm  
 The Altura Iliac Endograft length is standard across the product range at 65mm

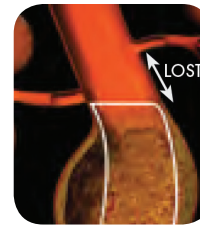

Traditional



Altura  
Gained Seal Zone

### Arterial Access


- Adequate iliac/femoral access that is compatible with vascular access techniques, devices, and/or accessories
- Ensure adequate femoral/iliac access that will accommodate a **14Fr** introducer sheath

### Aortic Neck

- Neck length required between renal arteries and aneurysm:
  - Length of  $\geq 15\text{mm}$
  - Inner wall diameter of **18-28mm**
  - Neck angulation of  $\leq 60^\circ$

### Iliac Arteries

- Distal common iliac landing zone:
  - Inner wall diameter of **8-18mm**
  - Length of  $\geq 15\text{mm}$

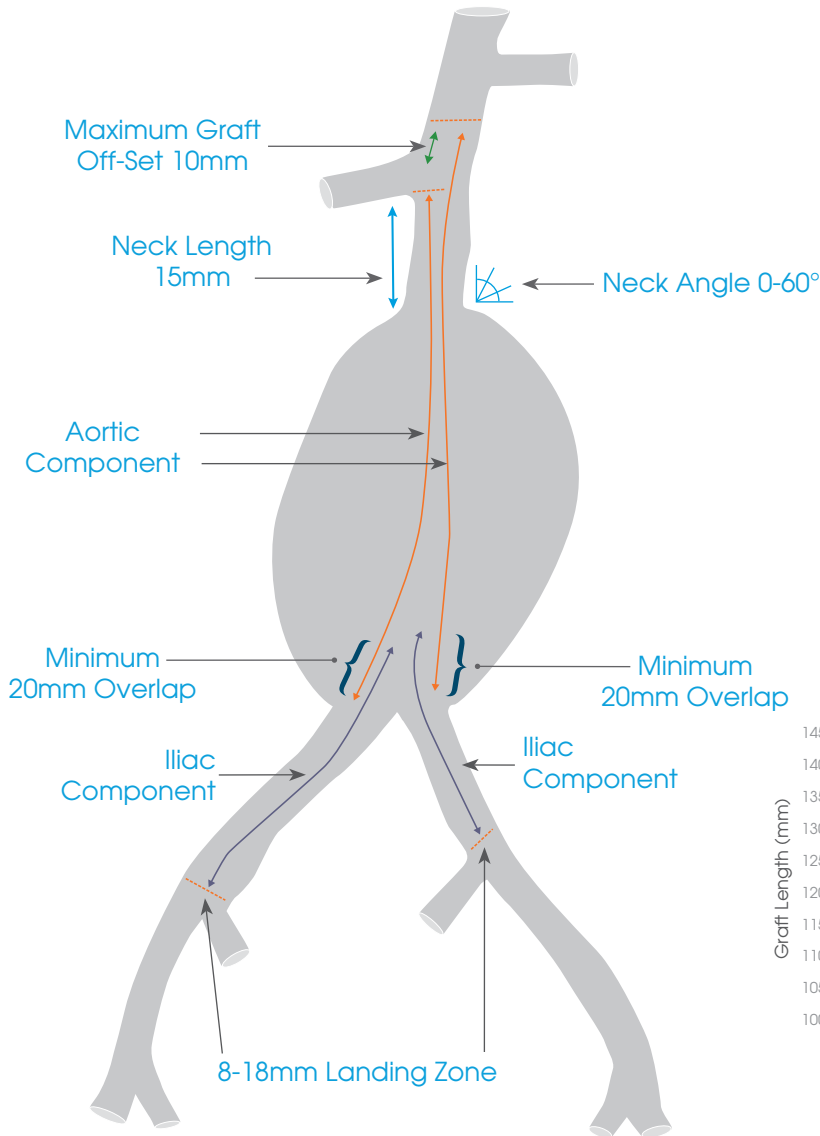

Please refer to Aortic and Iliac Endograft treatment ranges in order to select a treatment size

#### Corporate Office

Lombard Medical, Inc. | 6440 Oak Canyon, Suite 200  
Irvine, CA 92618 | Tel: 866 790 1447  
E-mail: USinfo@lombardmedical.com

#### United Kingdom

Lombard Medical, Inc. | Lombard Medical House  
4 Trident Park, Didcot | OX11 7HJ  
Tel: +44 (0) 1235 750800 | Email: globalinfo@lombardmedical.com

Refer to product IFU for full indications, contraindications, warnings and precautions.

Altura Endograft System is CE marked and not for distribution in the USA

